BEEKLEY MEDICAL[®] when your diagnosis must be right[®]

The Beekley Skin Marking System[®] for Mammography. Save *time*. Save *lives*.

Benefits

The Beekley Skin Marking System[®] for Mammography offers radiologists a simple and effective way to identify nipples, raised moles, surgical scars, palpable masses and non-palpable areas of concern or pain. As a result, interpreting mammograms may be faster and more accurate, helping you save more lives.

The system consists of five different Beekley Medical[®] skin markers, each with a distinctive and unmistakable shape to interpret on an image. A circle appearing on the image could only mean one thing – a raised mole. A line indicates an area of a previous surgery, a triangle symbolizes a palpable mass, a solid dot is used to mark a nipple and a square to mark non-palpable areas of concern or pain.

Breast imaging centers using the Beekley Skin Marking System for digital, analog and tomosynthesis mammograms report better communication, fewer questions, fewer additional views and permanent documentation.

Table of Contents



Beekley Medical® Nipple Markers (Pg. 2-3) Used as a reference to identify nipple location



Beekley Medical[®] Mole Markers (Pg. 4-5) Indicates if raised moles or nevi are present




Beekley Medical[®] Scar Markers (Pg. 6-7) Clearly defines a scar from a previous surgery



Beekley Medical® Palpable Mass Markers (Pg. 8-9) Identifies the location of a palpable abnormality



Beekley Medical[®] Non-Palpable Area of Concern or Pain Markers (Pg. 10-11) Communicates an area of concern or focal pain

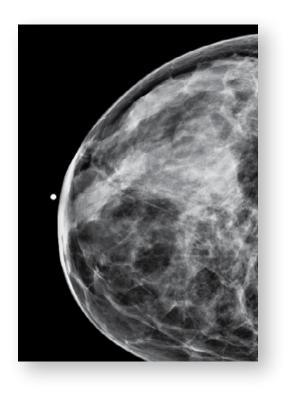

Beekley Medical® Nipple Markers

How does a nipple marker improve accuracy when used on every single patient?

- A marked nipple acts as a landmark to help measure the nipple-to-lesion distance. This highlights the precise location of the suspected lesion.
- Promotes good positioning and verifies nipple is in profile.
- Makes certain the shadow of the nipple out of profile is not mistaken for an inframammary lesion.
- Aims to differentiate a cancer or area of concern located posterior to the nipple.
- Records and simplifies year-to-year comparisons of historical charts.




Beekley Medical[®] Mole Markers

How does a mole marker save time?

- The mole marker signals that the soft density highlighted by the ring corresponds to an identified mole that is not of concern.
- The soft tissue image of a raised mole is consistently captured within the center of the marker.
- Beekley Medical mole markers are offered in ultra-sheer and low density variants, allowing all tissue detail and calcifications to image within the shadow of the marker.
- The communication on the mammogram becomes a clear and permanent part of the patient's record when images are transferred to another location.


THE RAISED CIRCLE ONLY FROM BEEKLEY MEDICAL

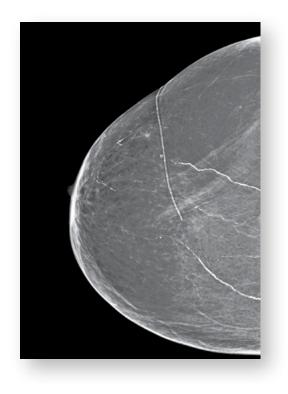

Beekley Medical[®] Scar Markers

How does a scar marker supplement a written history?

- Routine scar marking aims to eliminate confusion of where a biopsy was performed.
- The surgical history and mammographic findings are more easily correlated; distortion or asymmetry of the local breast architecture can be explained.
- Calcifications found in the area marked with a scar marker can be examined relative to their location at the biopsy site.
- Routine and consistent placement of the scar marker will assure that a reference point to the original surgery is maintained from year to year.

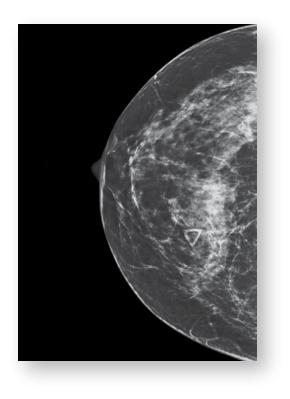


Beekley Medical[®] Palpable Mass Markers

How does a palpable mass marker remove doubt?

- The triangular shape (the international symbol for warning) immediately directs attention to the area of concern on the image without hiding or blending in with calcifications.
- A correlation is established between the physical findings of what is palpated in comparison to what is seen on the image.
- The presence of the palpable mass marker on the image provides documentation that the palpable abnormality is included on the film.
- Becomes a permanent record on the patient's image, documenting the area of concern.

THE RAISED TRIANGLE ONLY FROM BEEKLEY MEDICAL Beekley Medical[®] Non-Palpable Area of Concern or Pain Markers

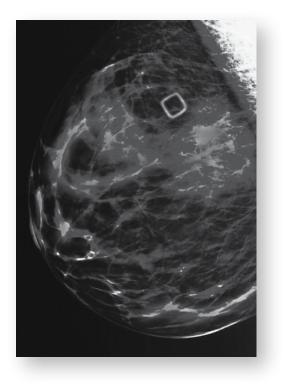

Beekley Medical[®] Non-Palpable Area of Concern or Pain Markers

Why use a square to mark non-palpable areas of concern or focal pain?

- Eliminate confusion on the area being marked with the new square shape.
- Provide permanent documentation since history charts can sometimes get lost.
- Facilitate faster read time with immediate identification of the area in question.

THE RAISED SQUARE ONLY FROM BEEKLEY MEDICAL Our products are backed by a 100% satisfaction guarantee. Returns and exchanges are accepted at anytime without penalty or restocking fees.

All Beekley Medical[®] products are lead-free, latex-free and hypoallergenic for your safety and that of your patients. Beekley Medical skin markers employ a medical grade adhesive for that "just right" stick — minimizing waste and resulting in overall lower cost.

We know your patient care relies on your order arriving when you need it and strive to always have your product in stock and delivered within 4 business days or less* upon receipt of your order.

Beekley Medical invests heavily in the research and development of new products to meet your needs and the demands of today's more sophisticated equipment and technologies such as markers specifically for full field digital mammography and digital breast tomosynthesis.

We proudly support the fight against breast cancer by donating a portion of the proceeds from Beekley's Mammography SPOT[®] skin markers to local and national breast cancer foundations.

^{*4} day guarantee applies to accounts within the U.S. using Beekley's preferred shipper. Other conditions may apply.

Call 1-800-233-5539 • Fax 1-800-735-1234 Visit www.beekley.com • Email info@beekley.com

Customers outside the U.S. – Contact your local distributor for pricing and product availability. To locate a distributor call +1-860-583-4700 or email international@beekley.com

Beekley Corporation, One Prestige Lane, Bristol, CT 06010 Tel: 1-800-233-5539 or +1-860-583-4700 Fax: 1-800-735-1234 www.beekley.com

BEEKLEY, BEEKLEY MEDICAL, BEEKLEY SKIN MARKING SYSTEM, SPOT, and WHEN YOUR DIAGNOSIS MUST BE RIGHT are Reg. U.S. Pat. & Tm. Off. BEEKLEY and SPOT are Registered Community Trademarks. Beekley is a registered trademark in Canada.



© 2015 Beekley Corporation. All rights reserved.